Rollator Diamond Deluxe

mit abnehm - und schwenkbaren Fußstützen, Stockhalter
with removable and swivelling footrests, cane holder
avec repose-pieds amovibles et pivotants, porte-canne
met afneembare en zwenkbare voetsteunen, stokhouder

Art.Nr. / REF 745R

CE

Teile des Rollator –
 Parts of walking device –
Pièces du Rollator
 Onderdelen van de rollator

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- 1 Adjustable tube of handgrips
- 2 Brake
- 3 Backrest
- 4 Seat
- 5 Rear wheel
- 6 Front wheel
- 7 Bag
- 8 Footrest
- 9 Safety screw Footrest
- 10 Cane holder

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Walking device for indoor and outdoor use for compensation and improvement respectively of restricted ability of walking. Can be used also as a transport chair because of the removable and swivelling footrests.

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This product is suitable for the reapplication. With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

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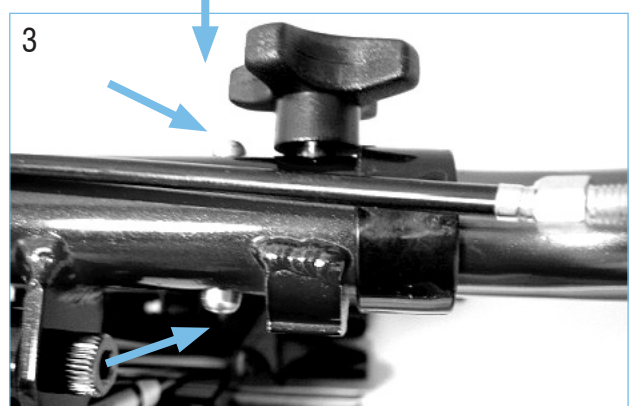
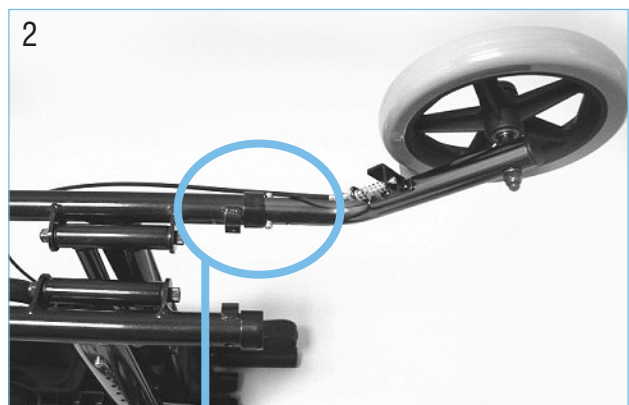
Prepare the individual parts as shown in picture 1. Insert the back wheels in the admission pipes (picture 2) and protect them with the protection screws (picture 3). The front wheels must be pushed powerfully in the admissions (picture 4). To reassemble press the red button and pull the wheel out (picture 5).

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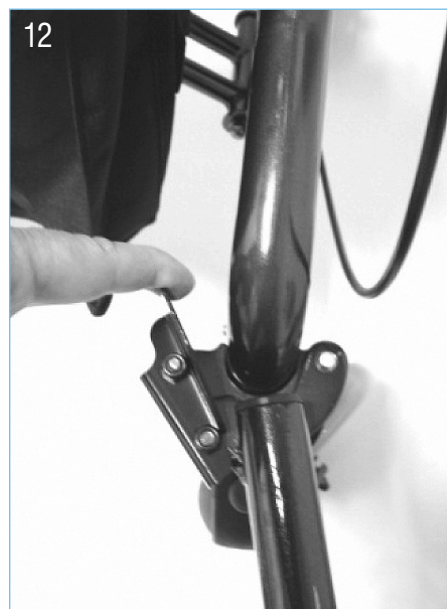
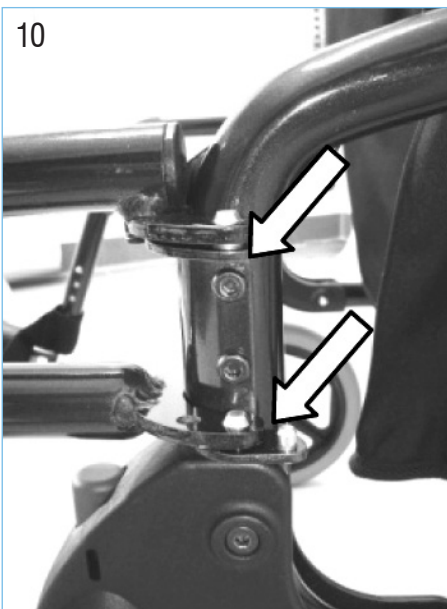
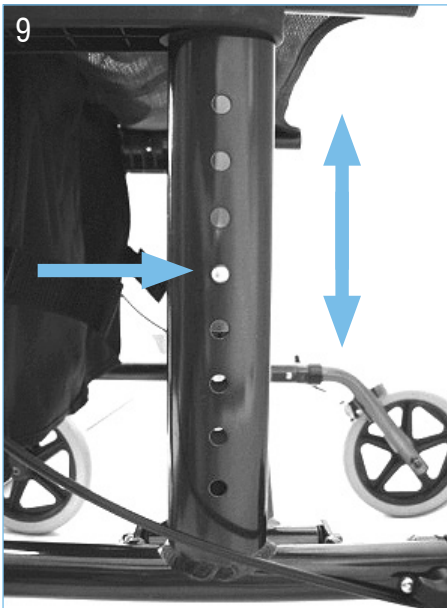
Put now the Rollator (picture 6) on the wheels (picture 7) and press from above at the side of the seat until it locks audible and perceptibly (picture 8). For height adjustment of the handgrips press both clips and shift the leadership pipe in the desired position (picture 9).

Hang up the footrests in such a way that it engages in both bolts (picture 10, 11). For swing away the footrests unlock the interlock as shown in picture 12. If you do not use the footrests, hang up this as shown in picture

13 in such a way that all 3 bolts have engaged, then the footrests are firmly arrested at the Rollator (picture 14).

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For parking the walking device press down the brakes until they click in the parking position (see picture). For unlocking the brakes pull them upwards towards the handgrips until they are unlocked (both brakes are in a middle position).

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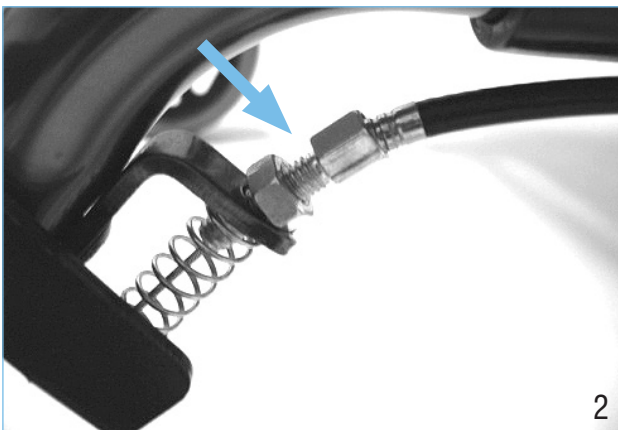
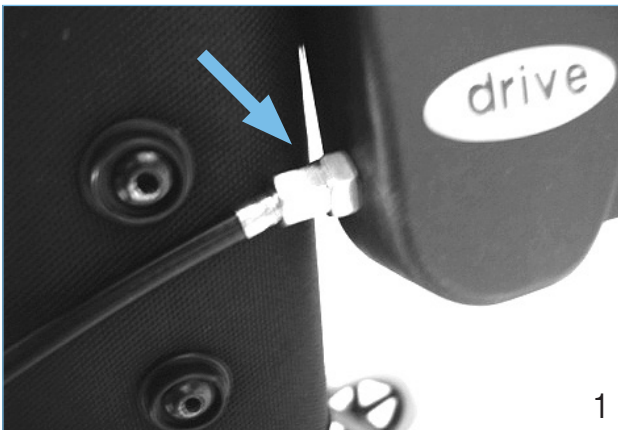
The effect of the brakes can be affected by abrasion of the wheels. Therefore test regularly the brakes.

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For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 1) or with the suitable tool (not included) on the brake wire (picture 2).

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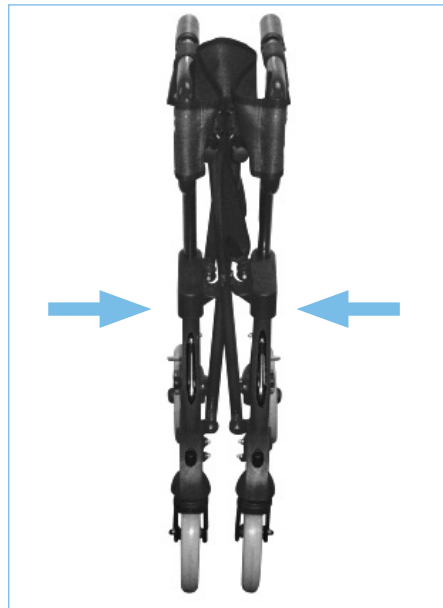
For folding pull the handle at the seat upwards and fold the rollator sidewise

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- 1 Product name
- 2 Article number
- 3 Serial number
- 4 Date of manufacturing year/week
- 5 Attention: See operating instructions

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